AVALON LABORATORIES

AVALON ELITE®

Vascular Access Kit

Instructions for Use
Vascular Access Kit

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 Needle/Nadel/Aiguille/Aguja/Ago/Nål/Kanyle/Naald/Agulha

 Scalpel/Skalpell/Scalpel/Bisturi/Scalpello/Skalpell/Skalpel/Scalpel/Bisturi

 Syringe/Spritze/Seringue/Jeringa/Siringa/Spruta/Sprøjte/Spuit/Seringa

General Statement

Federal (U.S.A.) law restricts this device to sale, distribution, and use by or on the order of a physician.

Note: Any medical procedures or techniques described in this document are meant for informational purposes only and should not replace the expertise of the physician and/or medical protocols.

Indication for Use

The Avalon Elite® Vascular Access Kit is intended for single use by trained physicians to assist in vessel cannulation.

Device Description

Vascular access kit (all contents are STERILE (EtO), NON-PYROGENIC, single use only).

Contents Include: (One set)

REF: 12100

- 1 – Guide Wire 0.038” (0.97 mm) x 100 cm (39.4”)
- 1 – Vessel Dilator 10 Fr.
- 1 – Vessel Dilator 12 Fr.
- 1 – Vessel Dilator 16 Fr.
- 1 – Vessel Dilator 20 Fr.
- 1 – Vessel Dilator 24 Fr.
- 1 – Introducer Needle – 18 Gauge
- 1 – Mini Scalpel – #11 Blade
- 1 – Syringe – 10 cc

REF: 12210

- 1 – Guide Wire 0.038” (0.97 mm) x 210 cm (82.7”)
- 1 – Vessel Dilator 10 Fr.
- 1 – Vessel Dilator 12 Fr.
- 1 – Vessel Dilator 16 Fr.
- 1 – Vessel Dilator 20 Fr.
- 1 – Vessel Dilator 24 Fr.
- 1 – Vessel Dilator 26 Fr.
- 1 – Vessel Dilator 30 Fr.
- 1 – Introducer Needle – 18 Gauge
- 1 – Mini Scalpel – #11 Blade
- 1 – Syringe – 10 cc

This device is not MRI compatible.

Instructions for Use

1. Check packaging and inspect all device components prior to use.

Warning: Do not use if package is open or damaged.

Caution: Assure that needle and scalpel blade protectors are in place before handling to avoid injury to user.

2. Follow an aseptic technique to remove the kit from packaging and transfer to the sterile field.
3. Using the introducer needle puncture the vessel to be dilated.
4. Depending on physician preference the scalpel can be used to visually expose the targeted vessel either before or after Step 3.
5. Insert the guide wire through the needle into the vessel.
6. Remove the needle while retaining the guide wire in the vessel.
Caution: To prevent damaging the guide wire, do not withdraw the guide wire prior to removing the needle from the vessel.
7. Insert the dilator(s) over the 0.038" (0.97 mm) guide wire into the vessel.
Caution: Always use the dilators over a guide wire.
8. Dispose of used products in accordance with established hospital protocol for biohazard and sharps waste.
Warning: The device is intended for single use only. Do not resterilize or reprocess as this can damage the device. Do not reuse as this could injure the patient and/or cause communication of infectious agents.

Contraindications

The physician is responsible to determine any patient conditions which may prohibit the safe use of the device. Use only as indicated.

Additional Precautions and Warnings

- Only physicians trained and experienced with vessel cannulation should use this device.
- Patient and procedure selection is the responsibility of the physician.
- Do not modify the device in any way.

Avalon Laboratories, LLC
Limited Warranty and Disclaimer

Avalon Laboratories LLC (“Avalon”) warrants that each product (“Product”) it manufactures will be free from defects in materials or workmanship prior to its “Use By Date.” Avalon’s sole liability shall be limited to replacement of, or credit for, any Product returned to Avalon, upon determination by Avalon, in its sole discretion, to have been defective in materials or workmanship. To qualify for this limited warranty, the Product must be returned to Avalon within forty-five (45) days after usage of the Product and must not have been altered or subjected to misuse, abuse or accident. In addition, the Product must be used in accordance with the labeling and instructions for use provided with the Product.

Except as expressly set forth above, Avalon disclaims all other warranties, with respect to the Product, including but not limited to any IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. Avalon shall not be liable to any person or entity for any medical expenses or damages caused by any use, defect, failure or malfunction of the product, whether a claim for such damages is based upon warranty, contract, tort or otherwise. AVALON IS NOT RESPONSIBLE FOR ANY DIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES BASED ON ANY DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT.

This Agreement shall be governed by and construed in accordance with the laws of the State of Delaware without regard to any conflict of law or choice of law rules or provisions.

In no event shall the United Nations Convention on the International Sale of Goods apply to this Agreement or to any transactions under the Agreement.

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AVALON LABORATORIES, LLC